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DEVELOPERS

Focus on your apps

Invest in app ops. Heroku handles the hard stuff — patching and upgrading, 24/7 ops and security, build systems, failovers, and more — so your developers can stay focused on building great apps.

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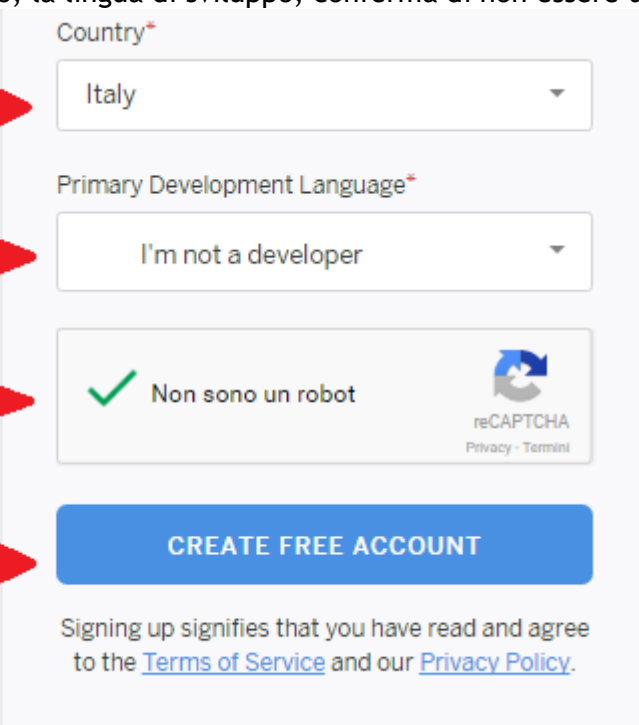
Country*
Italy

Primary Development Language*
I'm not a developer

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CREATE FREE ACCOUNT

Signing up signifies that you have read and agree to the [Terms of Service](#) and our [Privacy Policy](#).

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Thanks for signing up with Heroku! You must follow this link to activate your account:

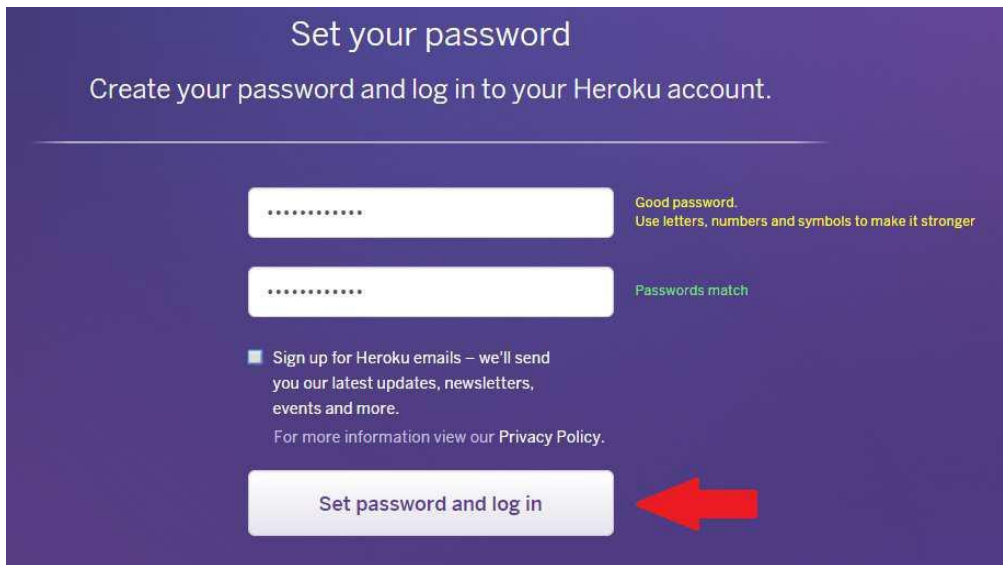
<https://id.heroku.com/account/accept/...>

Have fun, and don't hesitate to contact us with your feedback.

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Set your password

Create your password and log in to your Heroku account.

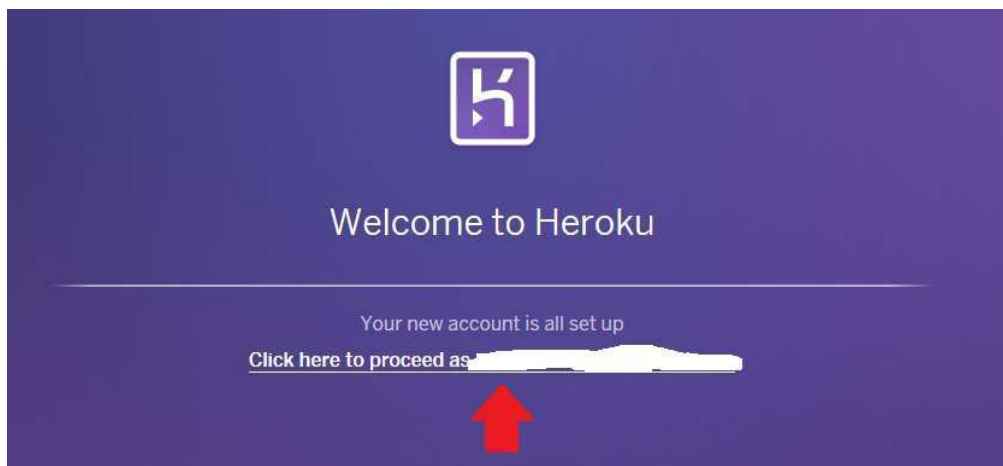
Good password.
Use letters, numbers and symbols to make it stronger

Passwords match

Sign up for Heroku emails – we'll send you our latest updates, newsletters, events and more.
For more information view our [Privacy Policy](#).

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
Nightscout Web Monitor (a.k.a. cgm-remote-monitor)

NIGHTSCOUT

#WeAreNotWaiting

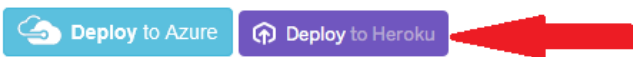build passing dependencies insecure coverage 75% codacy A Gitter [Join Chat](#) →

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2: selezionate 'Europe'

App Name (optional)

Leave blank and we'll choose one for you.

pippopluto is available

Runtime Selection

Your app can run in your choice of region in the Common Runtime.

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careportal rawbg iob cob cage sage bwp basal treatmentnotify food boluscalc bridge mmconnect pump temp basal openaps iage loop

*** MONGO_COLLECTION**

REQUIRED: The mongo collection used for CGM data. Default value is 'entries'. Most users should use the default.

*** API_SECRET**

REQUIRED: A secret passphrase that must be at least 12 characters long, required to enable POST and PUT; also required for the Care Portal

DISPLAY_UNITS

Choices: mg/dl and mmol. Setting to mmol puts the entire server into mmol mode by default, no further settings needed.

ENABLE

Used to enable optional features, expects a space delimited list, such as: careportal rawbg iob, see <https://github.com/nightscout/cgm-remote-monitor/blob/master/README.md> for more info

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Your account must be verified to create this app. Please add payment information to verify your account.

Credit Card Number

Card Number

Expiration

MM/YYYY

Security Code (CVV)

123

First Name

First Name

Last Name

Last Name

Billing Address

Address Line 1

Address Line 2

City

State/Province

ZIP/Postal Code

Country

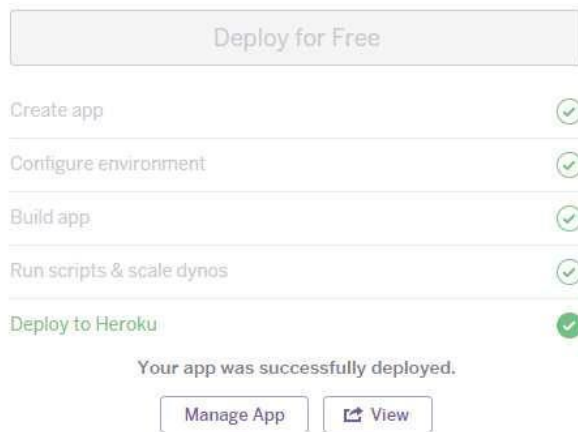
Additional Information (company name, tax ID, etc.)

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
Clicca su Manage App

Deploy to Heroku

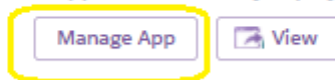Your app was successfully deployed.

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Overview Resources Deploy Metrics Activity Access **Settings**

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BRIDGE_SERVER	EU		
BRIDGE_USER_NAME	userdexcom		
BRIDGE_PASSWORD	passworddexcom		

Ora clicca su Deploy

Overview Resources **Deploy** Metrics Activity Access Settings

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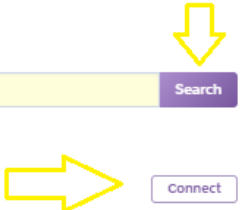
Clicca su connect.

Connect to GitHub

Connect this app to GitHub to enable code diffs and deploys.

Search for a repository to connect to

Missing a GitHub organization? [Ensure Heroku Dashboard has team access.](#)

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Deploy a GitHub branch

This will deploy the current state of the branch you specify below. [Learn more.](#)

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